# Caya<sup>®</sup> diaphragm gel

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PZN 02950119  
EAN 4013273001250  
CNK 2951168

# Caya<sup>®</sup> diaphragm gel

## Instructions for use (Please keep this leaflet!)

Caya<sup>®</sup> is a contraceptive gel for use with natural latex rubber or silicone diaphragms and cervical caps.

### Application

Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Caya<sup>®</sup> Gel in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap.

The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice.

The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperm – they are slowed in their forward movement or stopped.

### Composition

Water, lactic acid, sodium lactate, cellulose, sorbic acid

### Storage

Store in a dry place between 4°C and 25°C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

 Use within 3 months of opening.

### Side effects

Caya<sup>®</sup> Gel, because of its pH, is well tolerated by the vaginal flora. It can be used for as long and as often as required. If any problems arise when using Caya<sup>®</sup> Gel, please stop using it and consult your doctor or pharmacist.

### Contraindications

There have been no reports of women experiencing side effects during the use of Caya<sup>®</sup> Gel.

### Important

Caya<sup>®</sup> Gel offers **NO** protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. Caya<sup>®</sup>contoured diaphragm) or a cervical cap. Caya<sup>®</sup> Gel **IS NOT** a safe contraceptive method when used alone without a mechanical barrier. Caya<sup>®</sup> Gel is soluble in water and is compatible with natural latex rubber or polyurethane condoms.

### Use of vaginal applicator

The vaginal applicator is used optionally if additional gel is to be introduced in front of the already inserted diaphragm if intercourse is repeated.

1. Fill the applicator with Caya<sup>®</sup> Gel. Fit the applicator on the tube and carefully squeeze the gel out of the tube until the dosage mark is reached.
2. Remove the applicator from the tube.
3. Introduce the applicator, aperture first, as deeply as possible into the vagina.
4. As soon as you have correctly positioned the applicator, empty it by pressing gently on the plunger.
5. Remove the empty applicator from the vagina without pulling the plunger, since otherwise this would draw the gel back into the applicator barrel.
6. This vaginal applicator is reusable. However, it should be washed in warm water after every use.

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DeltaMed GmbH
Kessel GmbH